

Baby Band™ for Chin and Head

A more comfortable way to close the mouth
and position Baby Head Band™

respiralogics™

Baby Band for Chin and Head is designed to stabilize the position of the Baby Head Band and gently close the mouth during delivery of oxygen therapy, nCPAP or NIV therapy.

One Baby Band and three Hook Tabs are packaged per bag. Each Band is made of breathable, non-latex foam that absorbs and wicks away moisture, leaving the chin, cheeks, and head dry. The soft material of the Band conforms to the chin and head for a comfortable fit. One end of the Band and the Hook Tabs allow attachment to the Head Band for best placement.

The Baby Head Band for Chin and Head works with the Baby Head Band™ and Circuit Bumpers™. The Band can be used in all neonatal critical care units, delivery rooms, and special procedure units.


Ordering Information

| | | |
|-------|-----------------------------|---|
| 30510 | Baby Band for Chin and Head | 15 bags of 1 each Band with 3 Hook Tabs / box |
|-------|-----------------------------|---|


Companion Products

| | |
|-------|---|
| 30501 | Baby Head Band and Circuit Bumpers, Extra Small |
| 30502 | Baby Head Band and Circuit Bumpers, Small |
| 30503 | Baby Head Band and Circuit Bumpers, Medium |
| 30504 | Baby Head Band and Circuit Bumpers, Large |

Respiralogics products are available from Respiralogics and our specialty respiratory care distributors. For our latest product information visit www.respiralogics.com or email us at 4info@respiralogics.com.

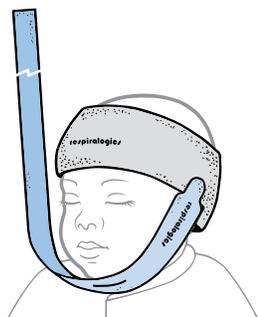

Center the slit in Baby Band under chin and secure Band to side of Baby Head Band

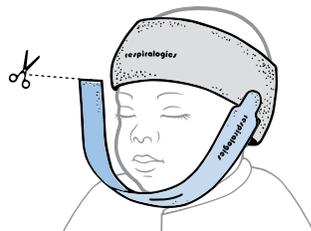

Trim Band, attach to Baby Head Band with Hook Tab on opposite side, above the ear.



Optional: attach trimmed band over the crown of the head using Hook Tabs.



FEATURES AND BENEFITS

Fashioned to provide a comfortable fit below the chin or around the head

Soft material to provide patient comfort

Allows caregiver to determine need to close the mouth or position the Head Band

Hook ends allow secure attachment to Baby Head Band

18-inch length of Baby Band can be customized to fit the infant's need

Single patient use, individually packaged and easy to apply

Respiralogics is a Global Respiratory Solutions, Inc. Company
3545 Airway Drive, Suite 104 • Reno, NV 89511 • 775 954 0160 • www.respiralogics.com

© 2026 Respiralogics. All rights reserved. Respiralogics, the Respiralogics logo, Baby Head Band and Circuit Bumpers and Baby Band for Chin and Head are trademarks of Global Respiratory Solutions, Inc. C30510.3